

Lugol Solution 5%

MenidiMedica
Biotechnology Applications

REF

70900-100: 100 mL.
70900-200: 200 mL.
70900-500: 500 mL.
70900-1000: 1 lt.



24 months, storage at RT



Περιεχόμενο

Ιώδιο, Συντηρητικά, Ρυθμιστικό Διάλυμα

Περιγραφή

Διάλυμα Lugol

Τρόπος διάθεσης

Γυάλινες φιάλες των 100, 200, 500, 1000 mL.

Δράση & χρήση

Το Lugol Solution 5% είναι ένα μέσο διαφοροποίησης των φυσιολογικών από τους επίφοβους ιστούς. Ο φυσιολογικός ιστός βάφεται καφέ λόγω της υψηλής περιεκτικότητας γλυκογόνου, ενώ ο ιστός που είναι ύποπτος για δυσπλασίες δεν βάφεται, και έτσι εμφανίζεται ωχρός σε σύγκριση με τον περιβάλλοντα ιστό. Μετά τη χρήση πλύνετε με νερό ή φυσιολογικό ορό.

Προειδοποιήσεις

Μόνο για Εξωτερική Χρήση. Σε περίπτωση ακούσιας οφθαλμικής επαφής, τα μάτια θα πρέπει να πλυθούν αμέσως με άφθονο νερό ή φυσιολογικό ορό, σηκώνοντας κατά διαστήματα τα άνω και κάτω βλέφαρα μέχρις ότου δεν υπάρχει καμία εμφάνιση υπολειμμάτων του διαλύματος (περίπου 15-20 λεπτά).

Αποθήκευση

Διατηρείτε το καλά κλεισμένο και προστατευμένο από το φως. Αποθηκεύστε το σε θερμοκρασία δωματίου 15°-30°C.

Απόρριψη

Οι ανοιγμένες φιάλες με αχρησιμοποίητο μέρος του προϊόντος που περιέχουν υπολείμματα του προϊόντος θα πρέπει να τοποθετούνται σε ένα κατάλληλο, ξηρό περιέκτη, ώστε ν'απορριφθούν αργότερα σύμφωνα με τις τοπικές πρακτικές διαχείρισης επικίνδυνων αποβλήτων.



Content

Iodine, Preservatives, Buffer

Description

Lugol Solution

How Supplied

Glass bottles of 100, 200, 500, 1000 mL.

Action & Uses

LUGOL Solution 5% is an agent used for differentiating normal from suspicious tissues. Normal tissue stains brown due to its high glycogen content, while tissue suspicious for malformations does not stain, and thus appears pale compared to the surrounding tissue. Wash after use with water or normal saline.

Warnings

For External Use Only. Should inadvertent ocular administration occur, the eye(s) should be washed immediately with large amounts of water or normal saline, occasionally lifting the upper and lower lids until no evidence of solution remains (approximately 15-20 minutes).

Storage

Keep tightly closed and protect from light. DO NOT use if seal is broken. Store at room temperature 15°- 30°C.

Disposal

Opened containers with unused portions of product should be placed in a suitable, dry container for later disposal according to local hazardous waste practices.



MenidiMedica Biotech
Menidi Aetolias-Akarnanias
30016-Greece
+30 2681088000, +30 6937115868
menidimedica@gmail.com
www.menidimedica.g

